

BRUNEL ARTIFICIAL TEARS

PROPRIETARY NAME AND DOSAGE FORM:

BRUNEL ARTIFICIAL TEARS eye drop

COMPOSITION:

Each 1 ml contains:

Carboxymethylcellulose Sodium.....0.5% w/v

Sterile Aqueous Base.....q.s.

Preservative:

Stabilised Oxychloro Complex.....0.005% w/v

MEDICAL DEVICE - GMDN CODE AND DESCRIPTION:

44237 – Eye lubricant.

ACTION:

BRUNEL ARTIFICIAL TEARS is a lubricating formula similar to normal tears, containing a mild preservative. It has no pharmacological effect. Due to its high molecular weight, it is unlikely to penetrate the cornea.

INDICATIONS:

BRUNEL ARTIFICIAL TEARS is intended to provide supplemental lubrication and hydration to the eye, to relieve dry, tired and/or strained eyes resulting from environmental factors like pollution, computer use, air conditioning, the wind and sun or contact lens wear.

CONTRAINDICATIONS:

Hypersensitivity to any of the ingredients.

WARNINGS AND SPECIAL PRECAUTIONS:

- For external use only.
- If you experience any eye pain, continued redness / irritation of the eye or if any changes in vision occur, discontinue use and consult your doctor or other healthcare professional.
- If irritation develops, discontinue contact lens wear and consult your optometrist.
- To avoid contamination, do not touch the tip of the container to any surface and avoid direct contact with the eye.
- Concomitant ocular medication should be administered at least 15 minutes prior to using **BRUNEL ARTIFICIAL TEARS**.
- Do not use more than 30 days after opening.

INTERACTIONS:

None known.

PREGNANCY AND LACTATION:

Safety has not been established for use during pregnancy and lactation. Please consult your doctor, pharmacist or other healthcare professional for advice before using **BRUNEL ARTIFICIAL TEARS**.

DOSAGE AND DIRECTIONS FOR USE:

BRUNEL ARTIFICIAL TEARS may be used in conjunction with rigid gas-permeable and soft contact lenses.

Instill 1 – 2 drops in the affected eye(s) 4 times a day, or as needed.

To open:

1. Remove tamper-proof seal.
2. Turn screwed-on cap clock wise to pierce nozzle.
3. Remove cap and use.

SIDE EFFECTS:

The following side effects may occur with the use of **BRUNEL ARTIFICIAL TEARS**.

Eye disorders:

Frequent: Eyes burning, irritation or pruritus (itching).

Less frequent: Eyelid swelling (oedema), excessive watering of the eyes (epiphora), redness of the eyelid (erythema) or eye (conjunctival hyperaemia), eye pain, eye discharge.

KNOWN SYMPTOMS OF OVERDOSAGE AND PARTICULARS OF ITS TREATMENT:

(See "**WARNINGS AND SPECIAL PRECAUTIONS**" and "**SIDE EFFECTS**").

Discontinue use and consult a healthcare practitioner or other registered healthcare professional immediately.

IDENTIFICATION:

A clear, colourless solution.

PRESENTATION:

White plastic dropper bottle containing 10 ml.

STORAGE INSTRUCTIONS:

Store at or below 25 °C in a cool, dry place, protected from light.

Keep tightly closed.

KEEP OUT OF REACH OF CHILDREN.

NAME AND BUSINESS ADDRESS OF DISTRIBUTOR:

Brunel Laboratoria (Pty) Ltd

1 Van Tonder Street

Sunderland Ridge

Centurion

0157

info@brunel.co.za

www.brunel.co.za

DATE OF PUBLICATION OF THE LEAFLET:

February 2017

artdPI-2S